

CLEANING INSTRUCTIONS

Please find below cleaning instructions:

Cleaning for Multiple Users Warning:

If you are using the device on multiple users, discard and replace the bacteria filter each time the device is used on a different person

Warning: Nebulization or humidification can increase the resistance of breathing system filters and the operator must monitor the breathing system filter frequently for increased resistance and blockage to ensure the delivery of the therapeutic pressure.

Note: When using the device on multiple users, it is recommended to use the "Reset Data" option before each new user.

Refer to Reset Data under Device Settings of this guide for additional information.

If you are using the device and/or link module on multiple users, complete the following steps to clean the device and/or link module before each new user.

1. Before cleaning, unplug the device and remove the link module. Also remove the blue pollen filter and light-blue disposable ultra-fine filter (if using).
2. Clean the outside of the device and link module only. Use a cloth with one of the following cleaning agents to clean the exterior of the device and link module:
 - Mild Detergent
 - 70% Isopropyl Alcohol
 - DisCide Towelettes
 - 10% Chlorine Bleach solution
3. Allow the device and link module to dry completely before plugging in the power cord and re-inserting the filters.